

Extra Fastening Vest

INTENDED USE

The **Extra Fastening Vest** is designed to provide maximum safety and comfort to the patient.

Especially suitable for hospital areas where patients are constantly posted and lifted out of bed or wheelchair, also used in geriatrics and intensive care units.

Do not appropriate for agitated patients.



COMPOSITION

Vest body: Polyamide 100%

Straps: non woven Polyester 100%

INSTRUCTIONS FOR USE

- After choosing the appropriate size, put the vest on the patient, taking into account that the opening is in front.
- Cross the vest by front and move the right strip through the left slot.
- Follow the same procedure for the rear. Pass the right strip through the left slot.
- Knot the straps between themselves, both on the right side as the left side. So that the vest was adjusted to the body.
- Knot the straps are at the height of the neck and make sure the proper position of the vest.
- Depending on how it ties the 4 straps to be carried 3 different kind of subjections. See the different techniques of subjection in attached images.



This technique involves setting the straps laterally to the bars of the bed. Allows the patient to be able to sit on the bed being securely attached.



2 of the straps hold the upper parts of the bed or wheelchair and hold the other 2 straps in the lower parts. This technique prevents the patient from sliding.



This technique provides greater security because 2 of the straps are secured to the headboard and 2 on the bars of the bed. Avoiding that the patient takes wrong positions, especially when they are seated. Especially suitable for use in wheelchairs.

PRODUCT CHARACTERISTICS

Versatility in implementation, adapting to each patient. Comfort for nurses and patients.
 Easy to apply, adjustable. Adapting to each patient.
 The back safety straps allows an additional patient adjustment. Increase security.
 The body vest allows air circulation. Fresh and comfortable.
 Extra-long straps to reach the frame of the bed. Flexibility in implementation.
 Polyester straps with additional sewn: strong product, durability. Reliable product.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.
 Do not use the product on injured skin.
 Do not use the product if it is pollutes, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.
 Shelf life: 5 years

UNIT PRODUCT MEASUREMENTS

REFERENCE	MODEL	L (cm)	W (cm)	STRAP L (cm)
1015500	SMALL	46.5	45	170
1015510	LARGE	61	54	170

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1015500	10 un/box	31x21x30	2.2
1015510	10 un/box	31x21x30	2.7

TOLERANCE: Weight ±10%

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

