

# Disposin one

## Disposable sponges without soap

**Disposin one** is an individual sponge of maximum softness that provides a great comfort of use.

Is a hygienic single-use sponge which is particularly suitable for personal hygiene and nursing care. The main feature is its softness when in contact with skin and the possibility of free choice of the gel as required by the patient's skin.

The product is specially destined to the care and cleaning of the skin, mainly, in people who require special attention.



### Product information

SPONGE: White polyurethane foam. Latex-free.

**DISPOSIN ONE**, with its **SINGLE USE** concept, radically eliminates any possibility of repeat infection due to the total certainty that it will not be used again.

This product is not incompatible with other products.

Weight	Unit size
15kg/m <sup>3</sup> ±5%	13x8x2.5 cm ±5%

### INSTRUCTIONS FOR USE

- Moisten the sponge **DISPOSIN ONE** and add the soap.
- Gently rub and lather the skin.
- Rinse with water

### Warnings and Recommendations

Keep out of the reach of children. Do not ingest.

Use properly sanitised equipment for each patient and wash. Use warm water for greater patient comfort.

Discard after use.

### Storage

Store in cool, dry place away from direct sunlight.

### PRESENTATION

REFERENCE	PACKAGING	UNITS PER PACK	BOX SIZE (cm)	BOX WEIGHT* (kg)
0005410	Multiple	1000 Units. (20 packs of 50 un.)	77x45x80 (sack)	4.0
0005410C	Individual	1000 Units. (20 packs of 50 un.)	50x70x80 (sack)	4.1
0005410D	Individual	2000 Units. (40 packs of 50 un.)	78x37x68.3	11.6
0005410E	Multiple	500 Units. (50 packs of 10 un.)	70x44x48	3.3
0005410F	Multiple	2000 Units. (40 packs of 50 un.)	78x29x66	10

\* TOLERANCE: Weight ± 0.5 Kg

### Complies with:

- Good Manufacturing Practices for Cosmetic Products according to ISO 22716.
- ISO 22716 and ISO 9001 certified company.

