

## Comfort Safety Limb Holder

### INTENDED USE


**Comfort Safety Limb Holder** is a medical device, which is intended to be used for the immobilization of the limbs, for use in emergency room beds, and anywhere you need a strong and safe subject, avoiding sudden movements. Suitable for restraining the wrists or ankles in patients who present state of agitation. Equipped with a soft pad with lengthy fastening strips.



### COMPOSITION

Interior part: Cotton 100%. Exterior part: velour. Polyurethane foam pad. Ring: Nylon 100%. Through adjustable Velcro closures. Strips of nonwoven polyester 100%.

### INSTRUCTIONS FOR USE

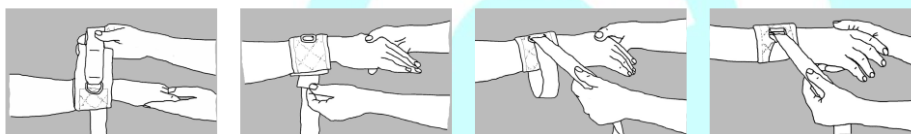

- Deploy the immobilizer and place around the patient's wrist or ankle.
- Adjust the immobilizer using the velcro system to the desired diameter.
- Passing the strings by clamping ring.
- Tie the strings at the metallic structure of the bed in such a way that the limb holder remains stretched.

### PRODUCT CHARACTERISTICS

Softness and commodity thanks to thicker cushioning. High resistance, allowing blood flow. Breathable, it prevents skin irritation, overheating, moisture and maceration.

### PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.  
Do not use the product on injured skin.  
Do not use the product if it is pollutes, dirty or evidently deteriorated.

### STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.  
Shelf-life: 5 years.

**UNIT PRODUCT MEASUREMENTS**

Anatomical design, adaptable to the patient. It can be used on the wrist or ankle:

REFERENCE	MODEL	LENGHT (cm)	WIDTH (cm)
1000202	LARGE	168	7.5
1000405	SMALL	135	5.5

CONFIGURATION			
REFERENCE	UNITS BOX	MESUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1000202	60 un/box	31x21x23	1.80
1000405	60 un/box	30x22x15	1.11

TOLERANCE: Weight  $\pm$  0.2 Kg

**IN COMPLIANCE WITH:**

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.



Sistema de Gestión  
ISO 9001:2015  
ISO 22716:2007  
www.tuv.com  
ID 9105078921