

Comfort Elbow-Heel Protector

INTENDED USE

Comfort Elbow-Heel Protector is a medical device indicated to avoid decubitus ulcers in elbow and heel on patients who must remain long periods in bed or chair with reduced mobility.

It can also be used as a wound dressing, as it is well placed and fixed, giving great comfort for its easy opening and closing after the cure.

Very effective product to protect the joints because the internal reinforcement spreads and distributes the pressure.



COMPOSITION

Inside: cotton

Outside and fastener strips: velours.

Padded with Polyurethane foam; anti-allergic and anti-mold.

INSTRUCTIONS FOR USE

- Remove the protector from its packaging.
- Place the protector at the base of the elbow or heel so that the pressure point matches with the internal reinforcement position.
- Adjust the straps with the velcro system.

PRODUCT CHARACTERISTICS

FEATURE	ADVANTAGES	PROFITS
Anatomic Design	Assures a perfect adjustment to both elbow and heel. Prevent the protector from moving or turning.	Placed directly under the point of pressure of elbow/heel, becomes very comfortable for the patient. Allow to reduce costs in user and inventory.
Material	High strength material interior and exterior.	Durability. Withstands greater number of washes.
Inner reinforcement	It has a small lift to the pressure point of the elbow/heel.	Softness and comfort for the patient.
Velcro system	Adjustable. Safety.	Flexibility in adjusting. Safety restraint.
Use for a single patient	One pair for each patient.	Eliminates the risk of cross-contamination.
1 wide strap	Adjusts patient protection preventing rotation.	Soft and lightweight. Comfort. Security.
Useful	Easy to apply. Minimal maintenance.	Save time nurses.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than a patient may provoke a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years.

UNIT PRODUCT MEASUREMENT

REFERENCE	MODEL	LENGHT (cm)	WIDTH (cm)	L. STRAP (cm)	W. STRAP (cm)
1000610	COMFORT	21	27	28	5

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX (kg)
1000610	60 un/box	40x25x40	1.88

TOLERANCE: Weight \pm 5% Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

