

Clavicle Splint

INTENDED USE

The **Clavicle Splint** is a medical device which is intended to be used for immobilize clavicles affected by injuries or fractures.

COMPOSITION

Base support: 100% cotton and Velours. Interior padding of polyurethane foam.

Strips: Velours, TNT 100% polyester and cotton 100%. Interior padding of polyurethane foam.

Rings: Nylon 100%



INSTRUCTIONS FOR USE

- Before placing the immobilizer must pass the tapes through the rings and adhere the velcro with the pads fitted for it.
- Put on the device like a vest.
- Adjust the immobilizer until it feels secure.

PRODUCT CHARACTERISTICS

It helps to maintain the proper posture.

Product very easy to adjust with the 4 set points.

It can be worn under clothing. The strips incorporate an axillary padding of polyurethane foam for greater patient comfort. It has 4 rings that facilitate its positioning, avoiding involuntary slips.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than a patient may provoke a cross-infection.

Do not use the product on injured skin.


Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years.

UNIT PRODUCT MEASUREMENTS

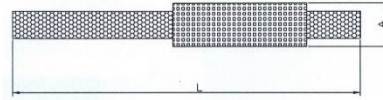

REFERENCE	MODEL	LENGHT (cm)	WIDTH (cm)	L. STRAP (cm)	W. STRAP (cm)
1000705	LARGE	16	11	108	5
1000706	MEDIUM	16	11	91	5
1000707	SMALL	16	11	80	5
1000711	EXTRA SMALL	16	11	72	5

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX (kg)
1000705	10 un/box	31x21x23	1.22
1000706	10 un/box	31x21x23	1.14
1000707	10 un/box	31x21x23	1.12
1000711	10 un/box	30x22x15	1.00

TOLERANCE: Weight ± 0.2 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

