

Bag for Ice

INTENDED USE

The **Bag for Ice** is a medical device, which is indicated to remain for extended periods on the area to be treated with cold therapy. More beneficial and comfortable for patients.

The longer duration of the ice inside the bag and the reduced frequency of ice replacement help optimize nursing staff time.

The four fastening straps make it easy to attach the bag to the treatment area.



COMPOSITION

The ice bag is made of a waterproof multilayer fabric inside and 2 layers of soft nonwoven fabric outside. The fastening strips are placed at the ends of the fabric and the clamp of the bag is stuck on top.

External side: smooth nonwoven fabric exempt of bleaching agents (external layer) waterproof nonwoven fabric (internal layer)

Internal side: waterproof plastic bag.

Fixing strips: nonwoven 100 % polyester.

Clamp: polypropylene.

INSTRUCTIONS FOR USE

- Take the clamp stuck on the ice bag.
- Fill the bag with 2/3 of ice and then remove remaining air.
- Collect the clamp below the bag neck. Avoid wrinkling of the folded area.
- Close the clamp ensuring its hermetic seal.
- Put the bag on the treatment area fixing the strips with knots.

PRODUCT CHARACTERISTICS

One of the main features of the product is its versatility to be used in any part of the body you may require. The material used for manufacturing this item, has the following features:

Softness and smoothness.

High resistance, which allows knot the strips with safety.

Breathable, avoiding skin irritations and moisture.

Easy placement at any part of patient's body.

Clamp closure makes it easy to refill the ice bag for repeated use on a single patient.

Latex free.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is pollutes, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years

UNIT PRODUCT MEASUREMENTS

REFERENCE	MODEL	BAG L (cm)	BAG W (cm)	TIE L (cm)	TIE W (cm)
1031600	SMALL	35	13.5	104	1.5
1031700	LARGE	36	18.5	104	1.5

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1031600	50 un/box	31x21x23	2.1
1031700	50 un/box	31x21x23	2.4

TOLERANCE: Weight ±10%

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

