

Braun Splint Cover

INTENDED USE

The **Braun Splint Cover** is a medical device, which is used to operate like a pad to prevent the bony prominences from causing ulcers by pressure.

COMPOSITION

Outer layer: Polyester mesh 100% and velours.

Inner layer: Padding with synthetic wadding 100% polyester, anti-allergic and anti-mould.

Back layer: Polyester mesh 100% and velours. Side reinforcement padding of high-density polyurethane foam



INSTRUCTIONS OF USE

- Remove the product from the packaging.
- Display the cover.
- Place the cover in the Braun splint and close it with velcro system.

CARACTERÍSTICAS

Breathable material that can be easily washed. Fresh and comfortable.

The Braun splint cover is designed with high quality padding to prevent the bony prominences from causing ulcers by pressure. Helps maintain ideal posture

It is very easy to adjust through a single adjustment point with Velcro® fastening system.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years

UNIT PRODUCT MEASUREMENTS

REFERENCE	MODEL	L (cm)	W (cm)
1040310	LARGE	105	54
1040305	MEDIUM	95	54
1040300	SMALL	85	54

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1000311	12 un/box	80x40x43	4.8
1000321	12 un/box	80x40x43	5.0
1000301	12 un/box	95x45x45	6.4

TOLERANCE: Weight \pm 0.2 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.



Sistema de
Gestión
ISO 9001:2015
ISO 22716:2007
www.tuv.com
ID: 9105078921