

Flexible Support

INTENDED USE

The **Flexible Support** is indicated to immobilize and protect the puncture sites of patients, used in resuscitation, emergencies, ICU...

It holds the intra-venous and intra-arterial therapy elements to the patient's limb, thus preventing any sudden movement from moving or pulling them out.

Device not suitable for agitated or aggressive patients or for patients with trauma in the area to be supported.



COMPOSITION

Support: Aluminium 100%

Support covering: Interior padding made of polyurethane foam with a layer of ivory coloured velour.

Straps: Velours and TNT Polyester 100%

INSTRUCTIONS FOR USE

- Place the bracket on the limb by bending it until it adapts.
- Adjust by inserting the straps around the limb and fastening with the velcro.

PRODUCT CHARACTERISTICS

Softness and commodity thanks to thicker cushioning. High resistance, allowing blood flow. Breathable, it prevents skin irritation, overheating, moisture and maceration.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is pollutes, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years

UNIT PRODUCT MEASUREMENTS

REFERENCE	MODEL	L (cm)	W (cm)
1060100	EXTRA SMALL	10	4
1060120	SMALL	16	6
1060125	MEDIUM	21	7
1060130	LARGE	26	8

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	BOX WEIGHT (kg)*
1060100	30 un/box	20x15x15	0.44
1060120	30 un/box	20x15x15	0.913
1060125	30 un/box	31x21x23	1.6
1060130	30 un/box	31x21x23	2.25

TOLERANCE: Weight ± 0.2 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

