

# Abdominal Support Belt

## INTENDED USE

The **Abdominal Support Belt** is designed as an alternative to the traditional patient support vest in bed- or chair-type structures.

Device intended exclusively for abdominal positioning. Do not apply to any other part of the body.

To keep a patient in a sitting position in a chair or bed and prevent possible falls or slips.



## COMPOSICIÓN

Internal-external layer: Polyamide 100%  
100% Polyurethane foam padding  
Straps: Cotton 100%

## INSTRUCTIONS FOR USE

- Place the belt over the patient's abdomen. The large model incorporates a velcro fastening system, so the abdominal belt can be fastened to the patient's lower back.
- Cross the strips over the lumbar area.
- Put the straps over the abdominal area, insert them through the respective side rings and make a safety knot. Take the straps back around to the back.
- Tie to the back of the wheelchair or bed.

## PRODUCT CHARACTERISTICS

It allows more stretch from the bottom and top of the product than from the centre. The product retains the shape without the need for uncomfortable buckles.

The strong straps give strength and durability to the product. Increases security.

Breathable material that can be easily washed. Fresh and comfortable.

It incorporates a cushion fresh and breathable. Patient comfort.

Extra long to be able to bind to the structure of the bed or wheelchair. Flexibility in implementation.

It incorporates a natural support without closer. Effective composition and patient comfort.

## PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is pollutes, dirty or evidently deteriorated.

## STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years

**UNIT PRODUCT MEASUREMENTS**

REFERENCE	MODEL	L (cm)	W (cm)	STRAP L (cm)
1070100	MEDIUM	70	17	147
1070150	LARGE	115	17	167

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX (kg)*
1070100	12 un/box	31x21x23	1.35
1070150	12 un/box	31x21x23	1.82

TOLERANCE: Weight ± 0.2 Kg

**IN COMPLIANCE WITH:**

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

