

# Perineal Body Belt

## INTENDED USE

The **Perineal Body Belt** is a medical device, which is designed as an alternative to the traditional vest subject to patients in beds, wheelchairs or geriatric chairs.

Device destined exclusively to the pelvic-abdominal positioning. Not to apply in any other part of the body.

Indicated to support the patient in wheelchairs incorporated and preventing possible falls or slides.



## COMPOSITION

Internal-external layer: Polyamide 100%  
100% Polyester mesh

100% Polyurethane foam padding  
Straps: Cotton 100%

## INSTRUCTIONS FOR USE

- Place the perineal belt on the wheelchair, so that the posterior part of the belt provided with the tapes stays on the support of the wheelchair.
- Sit the patient, who will remain placed on the product. Place the front part of the belt on the abdomen of the patient.
- Take the straps towards ahead, passing them across the respective lateral rings of retention and knot by means of a safety knot.
- Cross the straps to the back part of the support of the chair and knot.

## PRODUCT CHARACTERISTICS

It allows more stretch from the bottom and top of the product than from the centre. The product retains the shape without the need for uncomfortable buckles.

The strong straps give strength and durability to the product. Increases security.

Breathable material that can be easily washed. Fresh and comfortable.

It incorporates a cushion fresh and breathable. Patient comfort.

Extra long to be able to bind to the structure of the bed or wheelchair. Flexibility in implementation.

It incorporates a natural support without closer. Effective composition and patient comfort.

## PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is pollutes, dirty or evidently deteriorated.

## STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years

**UNIT PRODUCT MEASUREMENTS**

| REFERENCE | MODEL    | L (cm) | W (cm) | STRAP L (cm) |
|-----------|----------|--------|--------|--------------|
| 1070200   | STANDARD | 47     | 38     | 140          |

| CONFIGURATION |           |                       |                  |
|---------------|-----------|-----------------------|------------------|
| REFERENCE     | UNITS BOX | MEASUREMENTS BOX (cm) | WEIGHT BOX* (kg) |
| 1070200       | 12 un/box | 31x21x23              | 1.66             |

TOLERANCE: Weight  $\pm 0.2$  Kg

**IN COMPLIANCE WITH:**

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

