

# Operating Table Restraint

## INTENDED USE

The **Operating Table Restraint** is designed to secure patient to the operating table.

This device is designed for the trunk or abdominal area. Do not use on any other part of the body.

Used to keep a patient in a static position to avoid sudden patient movements during the operation.



## COMPOSITION

Internal-external layer: Polyester 100%

Ring: Nylon 100%

## INSTRUCTIONS FOR USE

- Unfold the strap over the area of the body and under the table to be held.
- Insert the end of the strap around the bracket and through the buckle.
- Adjust and close with velcro to secure.

## PRODUCT CHARACTERISTICS

Breathable material that can be washed easily. Fresh and comfortable.

The length to the diameter of each patient is adjustable by the combination of the velcro system, allowing an effective and effective support. Natural support.

Extra strong straps to increase safety during the surgical process.

## PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is pollutes, dirty or evidently deteriorated.

## STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.

Shelf life: 5 years

**UNIT PRODUCT MEASUREMENTS**

REFERENCE	MODEL	L (cm)	W (cm)
1070500	ARM / LEG	80	8
1070550	SURGICAL TABLE	150	6
1070560	SURGICAL TABLE	230	6

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	BOX WEIGHT* (kg)
1070500	20 un/box	21x15x15	0.56
1070550	20 un/box	21x15x15	0.56
1070560	20 un/box	21x15x15	0.56

TOLERANCE: Weight  $\pm 0.1$  Kg

**IN COMPLIANCE WITH:**

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.



Sistema de Gestión  
ISO 9001:2015  
ISO 22716:2007  
www.tuv.com  
ID 9105078921