

## Sportive Jockstrap

### INTENDED USE

The **Sportive Jockstrap** is a medical device, which is intended to be use as a prophylactic. It is recommended for any vigorous physical activity in which the male genital area is at risk, such as a sprain or bump.

### COMPOSITION

Jockstrap (Latex Free): 70% Polyester 30% Lycra



### INSTRUCTIONS FOR USE

- Measure the waist circumference to choose the required size.
- Put it on like a slip and lift it up until the elastic band reaches waist height and the bag lifts the genitals as much as is comfortable

### PRODUCT CHARACTERISTICS

Elastic band of great comfort and greater fixation. The bag prevents friction and moisture. Easy washing.



### PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.  
Do not use the product on injured skin.  
Do not use the product if it is pollutes, dirty or evidently deteriorated.

### STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.  
Shelf life: 5 years.

**UNIT PRODUCT MEASUREMENTS**

It is important to measure the waist circumference to choose the most appropriate size. It must be remembered that a loose garment can reduce the therapeutic efficacy of the product and a too tight can negatively influence blood circulation or other diseases that we intend to treat.

REFERENCE	MODEL	WAIST CIRCUMFERENCE (cm)	L (cm)	W (cm)
1070801	T1	56/62	22	26
1070802	T2	63/70	22	29
1070803	T3	71/78	22	32
1070804	T4	79/86	22	35
1070805	T5	87/96	22	38
1070806	T6	97/110	22	42

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX* (kg)
107080X	10 units/box	18x17x9	0.4

\* TOLERANCE: Weight  $\pm$  0.1 Kg

**IN COMPLIANCE WITH:**

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

