

Medical Jockstrap

INTENDED USE

The **Medical Jockstrap** is a medical device for male genital area in patients with epididymitis, hydrocele, varicocele, haematocele, spermatocele or local physiological alterations, among other diseases.

Indicated for post-surgical support in hernias and other operations.

COMPOSITION

Genital bag: Cotton 100%.

Waist strap: Polyester.

Under buttock straps: Lycra.

Ring: Nylon.



INSTRUCTIONS FOR USE

- Check the dimensions of the bag to choose the necessary size.
- Slip it up like a slip and lift it up to the waist and the bag will hold your genitals as comfortable as possible.
- Slide the strap through the buckle until the jockstrap is securely fastened.
- Separate the ribbons under the buttocks left and right, placing them in the groin area.
- Cut the excess strap if desired.

PRODUCT CHARACTERISTICS

Long length strap for optimal fixing.
The cotton bag prevents friction and moisture.
Available in several sizes.
Easy washing.



PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.
Do not use the product on injured skin.
Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years.

UNIT PRODUCT MEASUREMENTS

It is important to measure the dimensions of the bag to choose the most appropriate size. It must be remembered that a loose garment can reduce the therapeutic efficacy of the product and a too tight can negatively influence blood circulation or other diseases that we intend to treat.

REFERENCE	MODEL	WAIST CIRCUMFERENCE (cm)	Bag L (cm)	Bag W (cm)
1070811	T1	117	10.6	10
1070812	T2	117	11.25	10.5
1070813	T3	117	12	11
1070814	T4	117	12.75	11.5
1070815	T5	117	13.75	12
1070816	T6	117	14.5	13
1070817	T7	117	15	13.5
1070818	T8	117	15.75	14.50
1070819	T9	117	16.5	15
1070820	T10	150	17.75	15.75
1070821	T11	150	18.5	16.75
1070822	T12	150	19.5	17.25
1070823	T13	150	20.5	17.75
1070824	T14	150	21.5	18.75

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1070811-824	10 units/box	18x17x9	0.6

* TOLERANCE: Weight ± 0.1 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

