

Elbow-Heel Protector

INTENDED USE

Comfort Elbow-Heel Protector is a medical device indicated to avoid decubitus ulcers in elbow and heel on patients who must remain long periods in bed or chair with reduced mobility.

It can also be used as a wound dressing, as it is well placed and fixed, giving great comfort for its easy opening and closing after the cure.

Very effective product to protect the joints because the internal reinforcement spreads and distributes the pressure.



COMPOSITION

Inner/Outer side: Polyester 100%.

Padded with synthetic wadding 100% Polyester, anti-allergic and anti-mold.

Fastening Straps: Polyamide.

INSTRUCTIONS FOR USE

- Remove the protector from its packaging.
- Place the protector at the base of the elbow or heel so that the pressure point matches with the internal reinforcement position.
- Adjust the straps with the velcro system.

PRODUCT CHARACTERISTICS

FEATURE	ADVANTAGES	PROFITS
Anatomic Design	Assures a perfect adjustment to both elbow and heel. Prevent the protector from moving or turning.	Placed directly under the point of pressure of elbow/heel, becomes very comfortable for the patient. Allow to reduce costs in user and inventory.
Three layer cushion	Presents a small elevation for point of pressure of foot / heel.	Softness and comfort for the patient.
Velcro system	Adjustable. Safety.	Flexibility in adjusting. Safety restraint.
Use for a single patient	One pair for each patient.	Eliminates the risk of cross-contamination.
1 wide strap	Adjusts patient protection preventing rotation.	Soft and lightweight. Comfort. Security.
Useful	Easy to apply. Minimal maintenance.	Save time nurses.

- 1080200: Standard.
- 1080210: Internal reinforcement zone in the elbow or heel area.
- 1080210B: Extension with strap for double subsection and maximum protection zone.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than a patient may provoke a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years.

UNIT PRODUCT MEASUREMENT

REFERENCE	MODEL	LENGHT (cm)	WIDTH (cm)	L. STRAP (cm)	W. STRAP (cm)	REINFORCED AREA (cm)
1080200	STANDARD	21	27	28	5	-
1080210	REINFORCED	21	27	28	5	18x15
1080210B	DOUBLE SUPPORT	21	27	28	5	18x15

CONFIGURATION			
REFERENCE	UNITS BOX	MEASUREMENTS BOX (cm)	WEIGHT BOX (kg)
1080200	36 un / box	31x21x31	1.2
1080210	36 un / box	31x21x31	1.43
1080210B	36 un / box	38x21x35	1.75

TOLERANCE: Weight ± 5%

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

