

Opened Elbow-Heel Protector

INTENDED USE

Opened Elbow-Heel Protector is a medical device indicated to avoid decubitus ulcers in elbow and heel on patients who must remain long periods in bed or chair with reduced mobility.

It can also be used as a wound dressing, as it is well placed and fixed, giving great comfort for its easy opening and closing after the cure.

Very effective product to protect the joints because the internal reinforcement spreads and distributes the pressure.



COMPOSITION

Inner/Outer side: Polyester 100%.

Padded with synthetic wadding 100% Polyester, anti-allergic and anti-mold.

Internal reinforcement with higher density wadding 100% Polyester, anti-allergic and anti-mold.

Fastening Straps: Polyamide.

INSTRUCTIONS FOR USE

- Remove the protector from its packaging.
- Place the protector at the base of the elbow or heel so that the pressure point matches with the internal reinforcement position.
- Adjust the straps with the velcro system.

PRODUCT CHARACTERISTICS

| FEATURE | ADVANTAGES | PROFITS |
|--------------------------|--|--|
| Anatomic Design | Assures a perfect adjustment to both elbow and to heel. Prevent the protector from moving or turning. | Placed directly under the point of pressure of elbow/heel, becomes very comfortable for the patient. Allow to reduce costs in user and inventory. |
| Three layer cushion | Presents a small elevation for the point of pressure of the elbow/heel. | Softness and comfort for the patient. |
| Velcro system | Adjustable. Safety. | Flexibility in adjusting. Safety restraint. |
| Use for a single patient | One pair for each patient. | Avoid the risk of cross-contamination. |
| 2 wide straps | Fixe the protector to the patient avoiding moving and turning | Soft and ultralight. Comfort. Security. |
| Useful | Easy to use. Minimum maintenance. | Time saving for nursing staff. |

PRECAUTIONS AND RECOMMENDATIONS

The use by more than a patient may provoke a cross-infection.

Do not use the product on injured skin.

Do not use the product if it is polluted, dirty or evidently deteriorated.

STORAGE AND SHELF LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years.

UNIT PRODUCT MEASUREMENT

| REFERENCE | MODEL | LENGHT (cm) | WIDTH (cm) | L. STRAP (cm) | W. STRAP (cm) |
|-----------|-------|-------------|------------|---------------|---------------|
| 1080280 | Large | 43 | 23 | 28 | 11 |
| 1080290 | Short | 28 | 23 | 25 | 4 |

| CONFIGURATION | | | |
|---------------|-----------|-----------------------|-----------------|
| REFERENCE | UNITS BOX | MEASUREMENTS BOX (cm) | WEIGHT BOX (kg) |
| 1080280 | 20 un/box | 40x25x40 | 1.93 |
| 1080290 | 40 un/box | 38x21x35 | 1.5 |

TOLERANCE: Weight ± 5% Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

