

Fast Opening Limb Holder

INTENDED USE

Fast opening limb holder is a medical device, which is intended to be use for the immobilization of the limbs, for use in emergency room beds, and anywhere you need a medium and safe subject, avoiding sudden movements. Equipped with a soft pad with long support bands.




COMPOSITION

Interior and exterior: nonwoven polyester 100%. Polyurethane foam pad. Through adjustable Velcro closures. Strips of nonwoven polyester 100%.

INSTRUCTIONS FOR USE

- Deploy the immobilizer and place around the patient's wrist or ankle.
- Adjust the immobilizer using the velcro system to the desired diameter.
- Tie the strings at the metallic structure of the bed in such a way that the limb holder remains stretched.

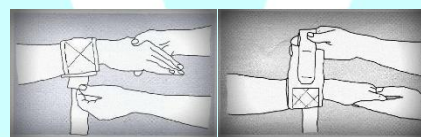

PRODUCT CHARACTERISTICS

Softness and commodity thanks to thicker cushioning. High resistance, allowing blood circulation. Breathable, it prevents skin irritations, overheating, humidity and maceration.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.
Do not use the product on injured skin.
Do not use the product if it is pollutes, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.
Shelf-life: 5 years.

UNIT PRODUCT MEASUREMENTS

Anatomical design, adaptable to the patient. It can be used on the wrist or ankle:

REFERENCE	LENGHT (cm)	WIDTH (cm)
1090110	168	7.5

CONFIGURATION			
REFERENCE	UNITS BOX	MESUREMENTS BOX (cm)	WEIGHT BOX* (kg)
1090110	60 un/box	31x21x31	1.68

TOLERANCE: Weight ± 0.2 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

