

Limb holder with Orifice

INDICATIONS

Limb Holder with Orifice is a medical device, which is intended to be use for the immobilization of the limbs, for use in emergency room beds and anywhere you need a medium and safe subject, avoiding sudden movements.

Anatomical design adjustable to be used in both, wrist or ankle. Provided with a soft padded wide area with two long straps for an easy fastening.



COMPOSITION

Interior and exterior: nonwoven polyester 100%. Polyurethane foam pad.

INSTRUCTIONS FOR USE



- Put limb holder around patient's limb with opening at external side. Place opposite string through opening
- Tie once more around limb and secure tie once again through opening.
- Take other strap and wrap around limb and place together. Both should have the same remaining length.
- Adjust and tie double knot at external side of limb, leaving about 2 cm of space between knot and limb.
- Attach to wheelchair, chair or bed.

PRODUCT CHARACTERISTICS

Softness. High resistance, allowing blood flow. Breathable, prevents skin irritation, suppresses overheating, moisture and maceration. Easy placement, indicated for routine use.

PRECAUTIONS AND RECOMMENDATIONS

The use by more than one patient may cause a cross-infection.
Do not use the product on injured skin.
Do not use the product if it is pollutes, dirty or evidently deteriorated.

STORAGE AND SHELF-LIFE

Keep away from sunlight. Keep dry.
Shelf life: 5 years.

UNIT PRODUCT MEASUREMENTS

Thanks to its anatomical design, the product is perfectly adapted to each patient and allowed to be used interchangeably to wrist or ankle.

REFERENCE	LENGHT (cm)	WIDTH (cm)
1090303	212	7.5

PRESENTATION			
REFERENCE	UNITS BOX	MESUREMENTS BOX (cm)	WEIGHT BOX (kg)
1090303	80	38x21x35	2.1

TOLERANCE: Weight ± 0.2 Kg

IN COMPLIANCE WITH:

- Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council.
- Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
- Enterprise certified by ISO 13485 and ISO 9001.
- Medical Devices Manufacturer Operating License No.: 3689-PS.

