

Dispobaño

Disposable soapy sponges

Dispobaño is a hygienic single-use and recyclable pre-soaped sponge which is particularly suitable for personal hygiene and nursing care, and is activated upon contact with water. The sponges are impregnated with a mild cleanser which helps maintain the skin's protective barrier and leaves a fresh, pleasant scent.

They are specially designed for gentle cleansing of the skin, mainly in persons who require nursing care.



Product information

SPONGE: Synthetic polyester fibre. Latex-free.

Weight	Unit size	
100 g/m² ±5%	20x12 cm ±5%	

SOAP: Skin cleansing gel, pH 5.5 ±0.5. Suitable for children.

Ingredients	Aqua, Sodium Laureth Sulfate, Sodium Chloride, Cocamide Dea, Cocamidopropyl Betaine,				
Potassium Sorbate, Sodium Benzoate, Citric Acid, Parfum, C.I. 61570, Linalool.					

Each sponge contains 2-3 g of soap before drying.

Microbiological quality control.

This gel is specially formulated with concentrated emollient ingredients to protect the skin. This may reduce foaming during use, but does not diminish its skin cleansing properties.

INSTRUCTIONS FOR USE













A few drops of water suffice to activate the cleanser in DISPOBAÑO sponges. DO NOT IMMERSE.

- · Gently rub and lather the skin.
- Rinse with water
- Dry skin thoroughly after washing.

Warnings and Recommendations

Keep out of the reach of children. Do not ingest. Avoid contact with eyes.

Use properly sanitised equipment for each patient and wash.

Use warm water for greater patient comfort.

Discard after use

Storage and shelf life

Store in cool, dry place away from direct sunlight.

Shelf life: Five years.

Rev.04 13/02/2019 PAGE. 1 / 2



PRESENTATION	PRESENTATION						
REFERENCE	UNITS PER PACK	UNITS PER BOX	BOX SIZE (cm)	BOX WEIGHT* (kg)			
0000100R	24 Units.	3,600 Units. 150 PACKS (6 BAGS OF 25 PACKS)	80x60x62	12.7			
0000100S	24 Units.	1,800 Units. 75 PACKS	76.5x62x46.5	7.3			
0000100L	24 Units.	2,400 Units. 100 PACKS (4 BAGS OF 25 PACKS)	60x40x44	7.8			

^{*} TOLERANCE: Weight ± 0.5 Kg

Individual pack bar code: EAN13 8427835100002

Complies with:

- Good Manufacturing Practices for Cosmetic Products according to ISO 22716.
- Regulation (EC) No 1223/2009 of the European Parliament and of the Council of 30 November 2009 on cosmetic products.
- Tested for paediatric use.
- Dermatologically tested.
- Gynecologically tested.
- ISO 22716 and ISO 9001 certified company.



C.V. MEDICA S.L. · C/ Ponent, 1 Pol. Ind. La Plana · 43424 Sarral – SPAIN · T.+34 977 890 598

www.cvmedica.com