

Dispofoam one aloe vera

Disposable soapy sponges with aloe vera extract

Dispofoam one aloe vera is an individual soapy sponge of maximum softness that provides a great comfort of use.

Is a hygienic single-use and pre-soaped sponge which is particularly suitable for personal hygiene and nursing care, and is activated upon contact with water. The sponges are impregnated with a mild cleanser with aloe vera extract which helps maintain the skin's protective barrier and leaves a fresh, pleasant scent.

They are specially designed for gentle cleansing of the skin, mainly in persons who require nursing care.



Product information

SPONGE: Polyurethane foam. Latex-free.

SOAP: Skin cleansing gel, pH 5.5 ± 0.5 . Suitable for children.

| Foam density | Unit size |
|-------------------------------|-----------------------|
| 15kg/m ³ $\pm 5\%$ | 13x8x2.5 cm $\pm 5\%$ |

| | |
|--------------------|---|
| Ingredients | Aqua, Sodium Laureth Sulfate, Sodium Chloride, Cocamide Dea, Cocamidopropyl Betaine, Potassium Sorbate, Sodium Benzoate, Citric Acid, Parfum, Aloe Barbadensis, C.I. 47005, C.I. 42090, Linalool. |
|--------------------|---|

Each sponge contains 2-3g of soap before drying.

Microbiological quality control.

This gel is specially formulated with concentrated emollient ingredients to protect the skin. This may reduce foaming during use, but does not diminish its skin cleansing properties.

INSTRUCTIONS FOR USE



A few drops of water suffice to activate the cleanser in **Dispofoam one aloe vera** sponges. **DO NOT IMMERSE.**

- Gently rub and lather the skin.
- Rinse.
- Dry skin thoroughly after washing.

Warnings and Recommendations

Keep out of the reach of children. Do not ingest. Avoid contact with eyes.

Use properly sanitised equipment for each patient and wash.

Use warm water for greater patient comfort.

Discard after use.

Storage and shelf life

Store in cool, dry place away from direct sunlight.

Shelf life: Five years.

| PRESENTATION | | | | |
|--------------|-----------------------|-------------------------------------|--------------------|------------------|
| REFERENCE | PACKAGING | UNITS PER BOX | BOX SIZE (cm) | BOX WEIGHT* (kg) |
| 0005405A | Individual compressed | 2000 Units. 40 Bags of 50 Units. | 78x37x68 | 13 |
| 0005405B | Individual | 500 Units. 10 Bags of 50 Units. | 70x44x48 | 4.2 |
| 0005405C | Individual | 1000 Units. 20 Bags of 50 Units. | 84x69x50 (sack) | 5.8 |

* TOLERANCE: Weight \pm 0.5 Kg

Individual pack bar code: EAN13 8427835100644

Complies with:

- Good Manufacturing Practices for Cosmetic Products according to ISO 22716.
- Regulation (EC) No 1223/2009 of the European Parliament and of the Council of 30 November 2009 on cosmetic products.
- Tested for paediatric use.
- Dermatologically tested.
- Gynecologically tested.
- ISO 22716 and ISO 9001 certified company.

