

DISPOMEDIC P SCRUB

SCRUB BRUSH with POVIDONE IODINE antiseptic for healthy skin-biocide for human hygiene

DESCRIPTION:

Brush/Sponge with **7.5% povidone iodine soap** suitable for antiseptic cleaning the hands of technical-healthcare personnel. Hand washing in high risk zones. Includes nail cleaner. Hygiene of preoperative patients.

COMPOSITION:

BRUSH: Polyethylene SPONGE: 100% polyurethane foam
 ANTISEPTIC: 7,5% povidone iodine soap (0.75% free iodine) 20 ml approx.
 NAIL CLEANER: Copolymer ABS (acrylonitrile butadiene styrene). Optional


INSTRUCTIONS:

For single use, for antiseptic cleaning of healthy skin. Pre-surgical use.
 Wash hands with water, apply antiseptic by brushing for 5 minutes and then rinse and dry with disposable towels or warm air. Causes serious eye irritation. Harmful to aquatic life with long lasting effects. For external use only. Avoid contact with eyes, mucous membranes and sensitive areas of the skin. Possible hypersensitivity to the skin. IF IN EYES: Rinse cautiously with water for several minutes. If you wear contact lenses, remove them if possible. Continue to rinse. Use with caution in children under 2 years of age. Do not use in persons sensitive to its components, nor for a long period of time. It should not be used together with products containing mercury. Do not use in newborns, pregnant women or during breast-feeding. Do not use in people with symptoms of thyroid disease. Keep out of the reach of children. IF SWALLOWED: If you feel unwell, contact a POISON CENTER (Tel.: +34 91 562 04 20) or a doctor. If it is necessary to consult a doctor, show the package or the label. To avoid risks to people and the environment, follow the instructions for use.

CHARACTERISTICS:

Brush with bristles that hold a soft sponge. Its anatomical design ensures it is more convenient and flexible for the user. The nail cleaner is a thin, green, solid stick, easy to use and with high tensile strength and tear.

PHYSICAL AND CHEMICAL PROPERTIES:

Appearance: liquid
 Colour: brown
 Density: 1.04 ± 0.035 g/ml

DIMENSIONS OF PRODUCT UNIT:
 80x50x37 mm. ±2.5 mm

PRESENTATION				
REFERENCE	DESCRIPTION	UNID/BOX	BOX MEASUREMENTS (cm)	BOX WEIGHT* (kg)
0030210E	DISPOMEDIC P SCRUB	100	38 x 21 x 35	3.7
0030210F	DISPOMEDIC P SCRUB with nail cleaner	100	38 x 21 x 35	4.0
0030210G	DISPOMEDIC P SCRUB 15 dispensers of 30 units	450	60 x 39,5 x 50,5 (dispenser 49,5 x 12.5 x 11)	16.8
0030210H	DISPOMEDIC P SCRUB with nail cleaner 15 dispensers of 30 units	450	60 x 39,5 x 50,5 (dispenser 49,5 x 12.5 x 11)	18.3

*TOLERANCE: Weight ± 0.5 Kg

COMPLIES WITH:

REGULATION (EU) No 528/2012 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 22 May 2012 concerning the making available on the market and use of biocidal products.

EN 12791:2005 Chemical disinfectants and antiseptics - Surgical hand disinfection - Test method and requirement (phase 2/step 2).

EN 13624:2014 Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of fungicidal or yeasticidal activity in the medical area - Test method and requirements (phase 2, step 1)

EN 1650:2008 Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of fungicidal or yeasticidal activity of chemical disinfectants and antiseptics used in food, industrial, domestic and institutional areas - Test method and requirements (phase 2, step 1).

EN 1499:1997 Chemical disinfectants and antiseptics - Hygienic handwash - Test method and requirements (phase 2/step 2).

EN 13727:2012+A1 Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of bactericidal activity in the medical area - Test method and requirements (phase 2, step 1)

ISO 9001 certified company.

Nº REG. AEMPS: 872-DES

